leitung **FR** Mode d'emploi de Cricath® Adult **IT** Istruzioni per l'uso di Cricath®  
Cricath® Adult **TR** Cricath® Adult kullanma talimatları **EN** Instructions for  
emploi de Cricath® **EN** Instructions for use Cricath® Adult **NL** Gebrui-  
**TR** Cricath® Adult ku**DE** Cricath® Adult Bedienungsanleitung Cricath® Adult  
Adult **IT** Istruzioni **FR** Mode d'emploi de Cricath® Adult instructie Cricath®  
lanma talimatları **IT** Istruzioni per l'uso di Cricath® Adult Cricath® Adult  
per l'uso di Cricath® **NL** Gebruiksaanstructie Cricath® Adult **PT** Instruções  
nstructions for use **PT** Instruções de utilização Cricath® Adult leitung **FR**  
t **NL** Gebruiksaan**TR** Cricath® Adult kullanma talimatları utilização Cricath®  
ricath® Adult **DE** Cricath® Adult Bedienungsanleitung **FR** Mode d'emploi de  
ructie Cricath® Adult **PT** Instruções de utilização Cricath® Adult **TR** Cricath®

September 2017

Instructions for use Cricath® Adult	4
Cricath® Adult Bedienungsanleitung	9
Mode d'emploi de Cricath® Adult	15
Istruzioni per l'uso di Cricath® Adult	21
Gebruiksaanwijzing Cricath® Adult	27
Instruções de utilização Cricath® Adult	33
Cricath® Adult kullanma talimatları	39

EN

DE

FR

IT

NL

PT

TR

Instruction for use Cricath Adult

Name Ventinova Medical B.V.

Product name	Cricath® Adult	Sterile	
Product logo		Single Use	
ID = Inner diameter	2.0 mm	Consult instructions for use	
Length	7 cm	Does not contain natural rubber latex	
Product number			

**Operator**

Cricath® Adult must be applied by, or under supervision of, medical personnel trained and experienced in airway management.

**Intended use**

Cricath® Adult is intended for obtaining access to the trachea when conventional intubation by mask and/or large-bore endotracheal tube cannot be performed in adult patients.

**Patient group**

Adults

**Contraindications**

None known

**Potential complications**

- Bleeding
- Hematoma
- Catheter dislodgement
- Tissue emphysema
- Pneumomediastinum
- Oesophageal puncture

**Product description**

Cricath® Adult is a cricothyrotomy catheter with a length of 7 cm and an inner diameter of 2.0 mm, introduced over a 1.8 mm needle.

**Packaging**

- The packaging contains Cricath® Adult including a syringe and neck strap.
- It is packed in a peel-open package sterilized by use of ethylene oxide.
- Cricath® Adult is sterile as long as the packaging is undamaged and unopened. Do not use Cricath® Adult if any doubt exists on the integrity of the packaging.
- Store the packaged product in a dry place.
- Avoid extended exposure to light.

## Instruction for use Cricath Adult

### Required additional material

- Sterile water
- Ventilation device with male Luer connector (e.g. Ventrain®)

### Warnings and precautions

- Use Cricath® Adult only in combination with ventilators that function properly when connected to a small bore catheter, for example Ventrain®. Please refer to the user instructions of the ventilation device.
- Secure Cricath® Adult firmly during use in order to keep the catheter in position within the trachea at all times.
- Do not reinsert the needle into the catheter once the needle is withdrawn. It may damage or puncture the catheter wall.
- If possible and appropriate aseptic technique and local anesthetic should be applied.

### Instructions for use

- 1 Remove the protection tube from the catheter, fill the syringe with sterile water or saline and attach it to the catheter.
- 2 Prepare the anterior neck of the patient.
- 3 Identify the cricothyroid membrane, located between the cricoid and thyroid cartilage, by palpation (*Figure A* and *A\**).
- 4 Stabilize the trachea between thumb and index finger.
- 5 Hold the catheter at the transparent needle hub or water-filled syringe.
- 6 Insert the catheter caudad by puncturing the cricothyroid membrane with the tip of the needle (*Figure A* and *A\**). When entering the trachea, a loss of resistance can be felt.

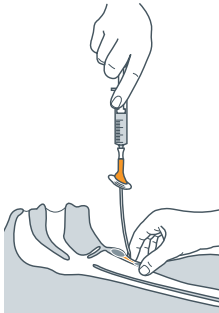
Figure A

Cricothyroid membrane

Figure A\*

- 7 Pull the plunger of the syringe to confirm the catheter position in the trachea (Figure B). Air should be aspirated.
- 8 Carefully advance only the catheter into the trachea caudad until the flange rests on the neck (Figure C).

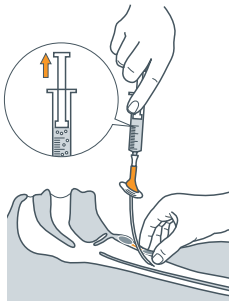
Figure B

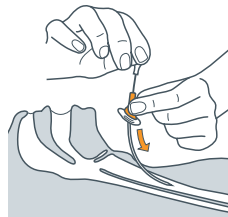
Figure C

## Instruction for use Cricath Adult

- 9 Then, remove the needle (*Figure D*). Do not reinsert the needle into the catheter once the needle is withdrawn. It may damage or puncture the catheter wall.
- 10 Attach the syringe to the catheter and aspirate again to confirm position of the catheter.
- 11 Secure the catheter with the supplied strap or other means (*Figure E*).

Figure D

Figure E

- 12 Connect an appropriate ventilation device (e.g. Ventrain®) to the catheter (*Figure F*).

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**Ventinova Medical B.V.** 5652 BJ Eindhoven **T** +31 (0)40 751 60 20